


MR Excite HD Quick Steps

PURE* (Phased Array Uniformity Enhancement)

TiP Training Choices

PURE Work Flow

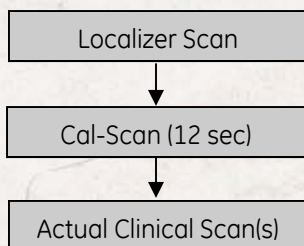

- PURE is also available under CLARIVIEW as a post-processing option.

Tips — If you would like to have two series with and without PURE applied, you MUST select “**none**” under the “**Image Enhance**” icon and use CLARIVIEW with

Cal-Scan Set-up¹

- Select coil “currently connected”
- Select scan mode = **Calib**²
- Select **8 mm** slice thickness (recommended)
- In Graphics Rx, cover the entire anatomy of interest and **add extra 3 slices max** in S and I directions³

Tips

- 1- No need to repeat the cal-scan for each series unless the landmark and/or coil location has changed.
- 2- PURE and ASSET share the same cal-scan.
- 3- If the clinical scan falls out of the cal-scan volume, that portion of the image will be cut off.

Compatibility

- PURE can be used with most receive-only PA coils, and 8-ch T/R MRID Knee Coil.
- PURE can be used with PROPELLER only as a post-processing option with CLARIVIEW.

**PURE is designed to reduce surface coil intensity variation through a calibration process.*

Where is the PURE option?

- Auto-PURE is available under the “Image Enhance” icon on the Scan Prescription page.

